




Instructions to Access Images and Reports:

1. Log In using your assigned *Username and Temporary Password* which will have been e-mailed to you in response to your application for access




2. You will be prompted to change your Temporary Password

3. Screen will appear

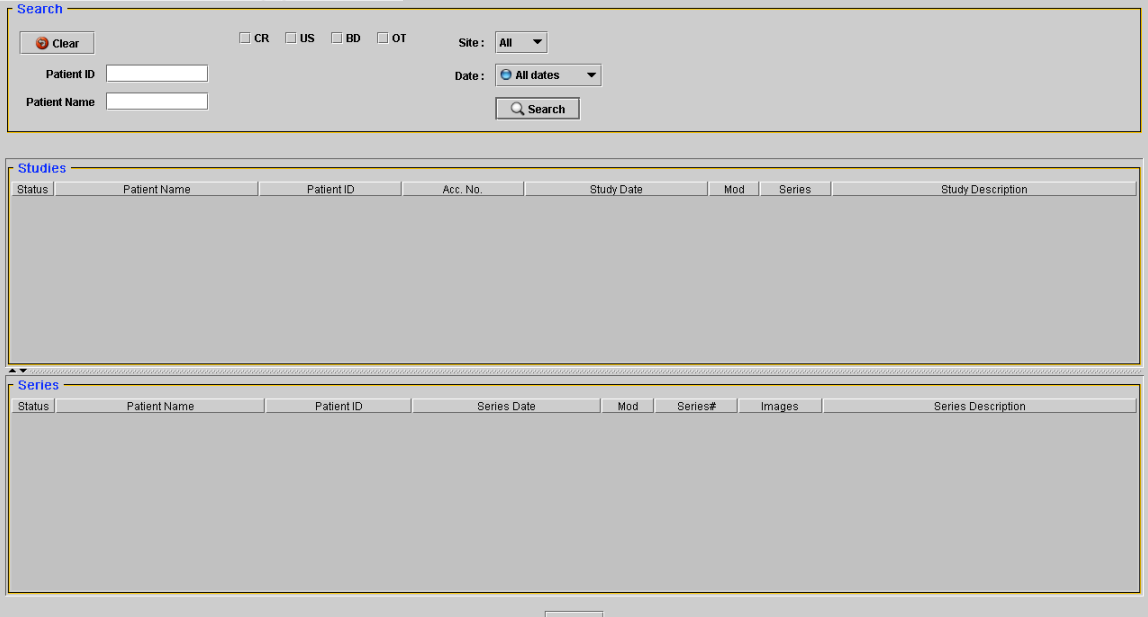

- (a) Enter patient name **xxxxxx**
- (b) Select Modality i.e. **CR for routine x-rays, US for Ultrasound etc**
- (c) Select from site drop down box **MRMI**
- (d) Select date range from **All Dates** drop down box

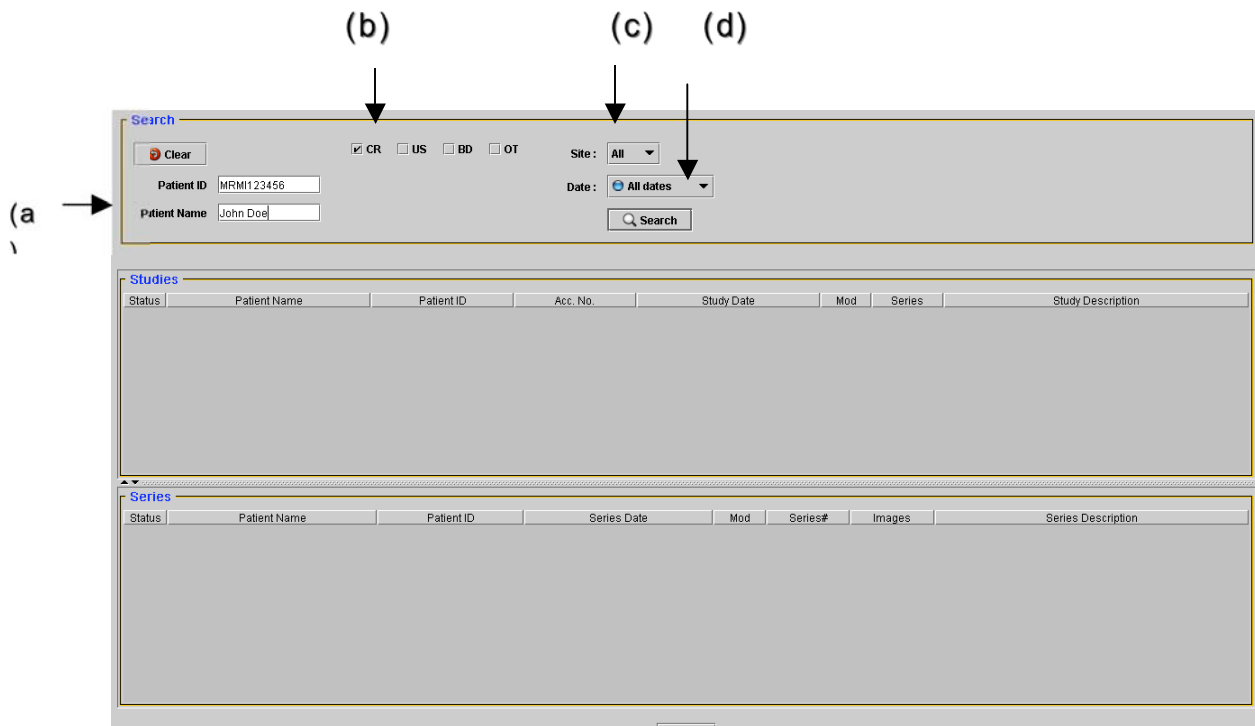

4. Hi light desired Study and Double Click Left Mouse or
5. Hi light desired Series and Double Click Left Mouse
 Images will appear as “thumbnails” below tool bar
 Page through images by using center wheel on mouse or
 Drag images from “thumbnail” display
6. To View Reports click on the 2nd Icon from the left located below the “print” selection on the tool bar.
7. For the **Complete Tool Set** right click on the mouse and select desired tool.